

Turn.aido Mattress Covers

Unique new Rotational Therapy System - Patient turns the easy way

- **Greater staff efficiency:** patient turns can now be executed quickly by a *single* nurse, with less effort, and consequently less risk of back strain.
- **'No Lifting Policy'** converts theory into practice by enabling carers or relatives to manoeuvre a patient without waking them up; minimises pain to the patient and reduces sleep disturbance.
- **Patient friendly-** less patient distress especially in the case of immobile or obese patients.



Note how the patient stays along the middle of the mattress. This is unique to **Turn.aido** mattress covers. On a normal mattress wide patients would come close to the edge when rolled on their side.

Full choice of 30°, 60°, 90°, 135° degrees of turn are available.

How it's done

1. Raise patient's furthest arm to head level.
2. Raise nearest knee to 90° flexion to help the rolling of the hips.
3. Release Velcro flaps on sides.
4. Grasp cotton sheet level with shoulders and hips.
5. Pull the sheet gently at 60° to rotate the patient to the desired angle of tilt – the integral glide membrane makes this action easy.

- *Turn.aido is especially valuable in cases of severe pain, bone metastasis and pressure ulcer therapy.*

Turn.aido  - fast and easy

First Technicare Limited
Unit 10, Acorn Production Centre
105 Blundell Street, London N7 9BN

Tel: 020 7609 8761 Fax: 020 7607 1062
Email: tripos3@aol.com
Web: www.firsttechnicare.com

FIRST
Technicare
Ltd

Innovation in pressure care



Turn.aido




•Velcro flaps are released to carry out a turn, then locked to secure the position. The glide motion is side to side only - easier turning encourages more frequent turning.



- Fast
- Easy glide
- Less sleep disruption
- Less discomfort for the patient being turned
- Carers can often carry out changes in position throughout the 24 hours rather than just relying on Community Nurse visits.



The Turn.aido principle

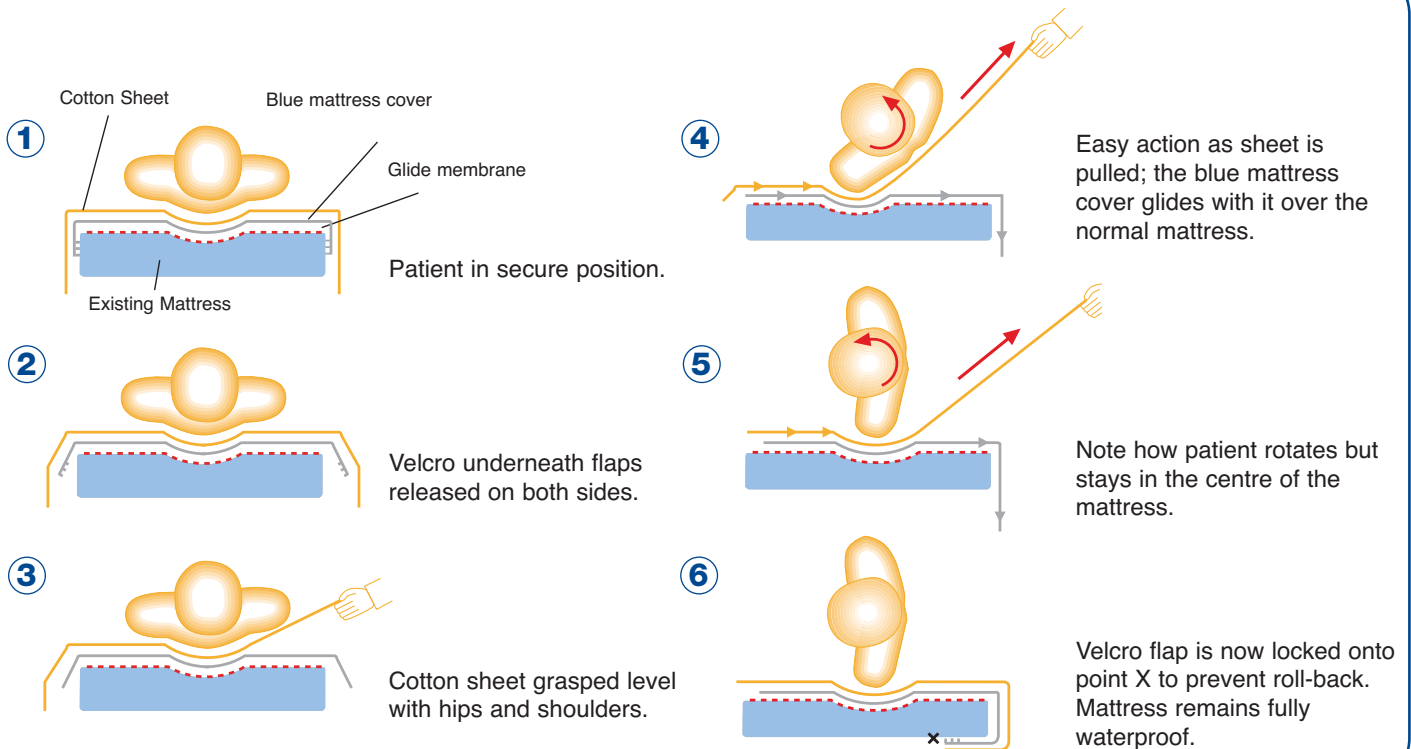

Turn.aido mattress covers are available in 2 sizes:

1) Fitted covers that slip over the mattress- Code T01a (76"x35"x6")(193.1 x88.9 x15.2cm) and Code T01b (80"x35"x6")(203.3 x88.9 x15.2cm)

2) They are also available as replacement covers for a standard foam mattress or to accommodate some types of thick overlays: *(Details available on request).*

Code To2a (76"x35"x6")(193.1 x88.9 x15.2cm) and Code To2b (80"x35"x6")(203.3 x88.9 x15.2cm)

Cleaning Instructions for Turn.aido Mattress Covers

The covers can be wiped down using a solution of neutral detergent and hot water applied with single use wipe or can be laundered at 60°c. Cleaning agents containing phenols and alcohols should not be used and the cover should be thoroughly drip-dried before being refitted to the mattress. Do not tumble dry. Do not dry clean. For heavy soiling, the cover should be laundered at 71°c with a dilute detergent solution.

Fire Retardancy: Complies with BS.EN 597-1&2 1995

Patent applications number 0103963.5.

CE Marking in accordance with 93/42/EEC

Class I low risk non active device